
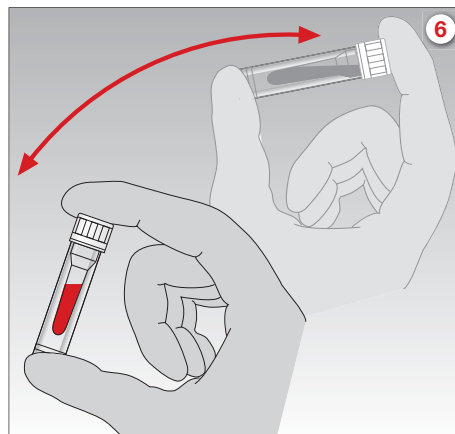
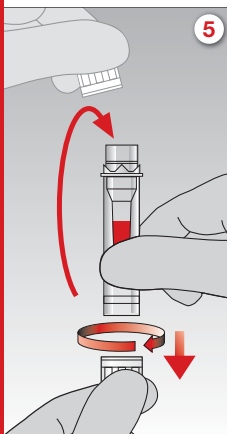
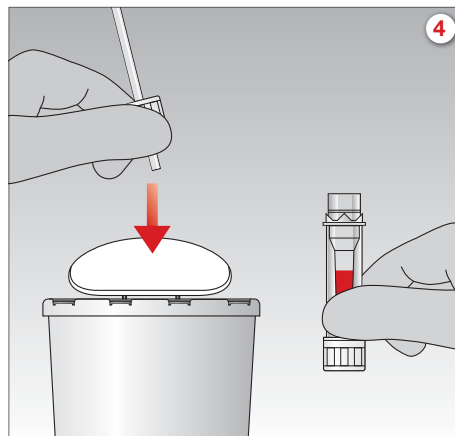
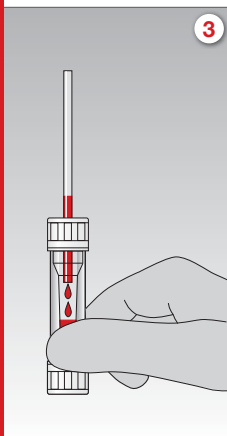
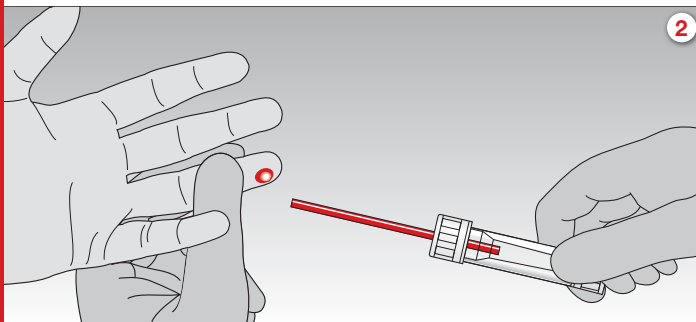


Microvette® 100/200

User Guide

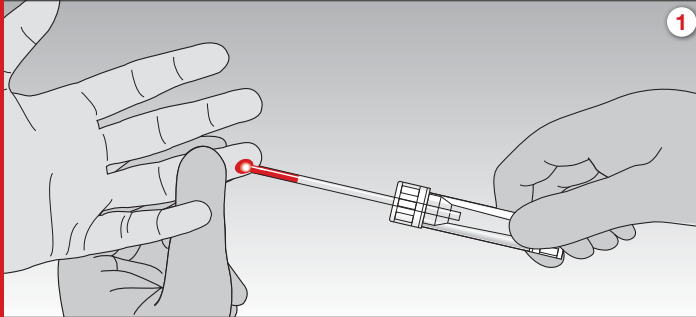


1. Collect blood with the assembled End-to-End 100 µl or 200 µl capillary. For collection, best results are achieved if the Microvette® is held in horizontal or slightly inclined position. During blood collection, hold the capillary 3 to 5 mm away from the puncture site.
2. Collection is complete when the End-to-End capillary is entirely filled with blood.
3. Hold the tube upright to allow the blood to flow from the capillary into the Microvette®.
4. Turn the cap to remove and discard the preassembled capillary as one unit.
5. Remove the cap from the base and seal the Microvette® ('click' position).
6. Mix sample thoroughly by inverting the Microvette®.

All patient blood specimens should be treated with standard precautions.


Wear gloves!

Microvette® 300/500


User Guide

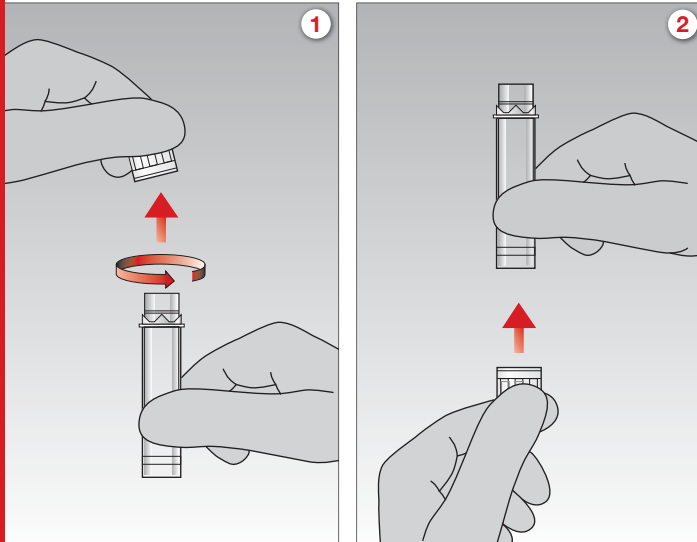


1. Turn the cap to remove it from the Microvette®.

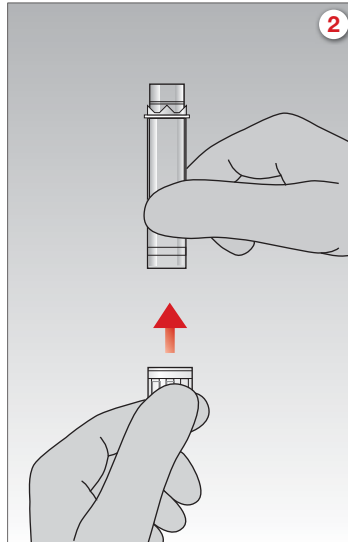


2. Attach the cap to the base of the Microvette®.

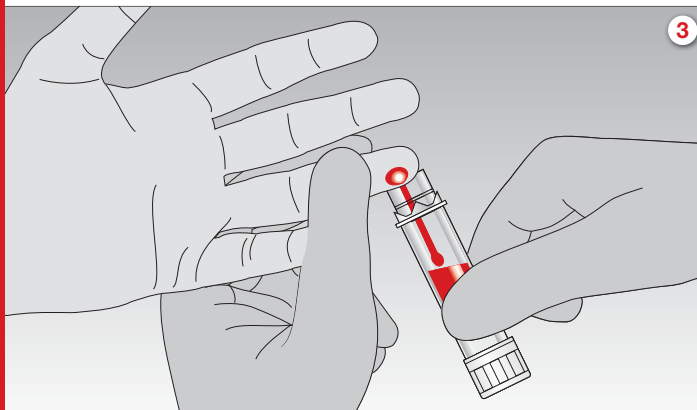

3. Collect blood by gravity-flow using any part of the collection rim.

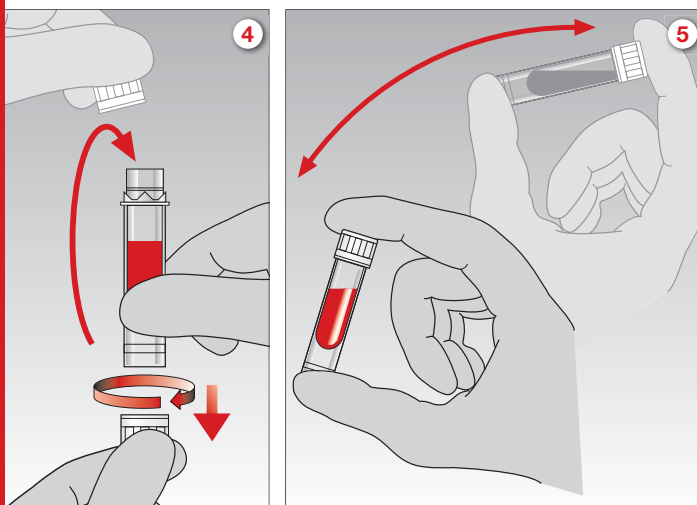

4. Remove the cap from the base and seal the Microvette® ('click' position).

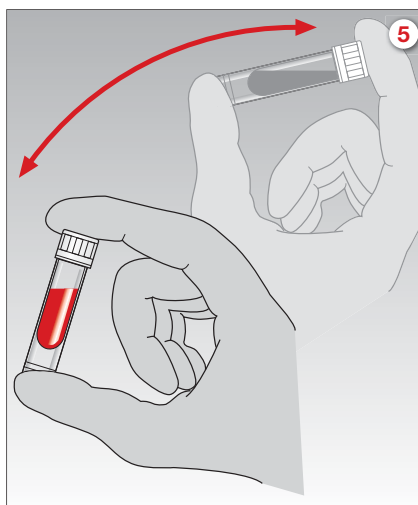

5. Mix sample thoroughly by inverting the Microvette®.

All patient blood specimens should be treated with standard precautions.

Wear gloves!

Technical modifications reserved

This document may contain information on products that may not be available in particular countries

264d-200

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 305-0
Fax +49 2293 305-3992
export@sarstedt.com
www.sarstedt.com

 **SARSTEDT**